# Neck Loop 2-channel, 80 cm


ENGLISH: Neck Loop, 2-channel.....page 3

DANSK: Hals Slyng, 2-kanals.....side 4

NEDERLANDS: Neklus, 2-kanaals.....Pag. 5

NORSK: Hals Slyng, 2-kanals.....side 6

SVENSKA: Hals slinga, 2-kanalig.....sid 7

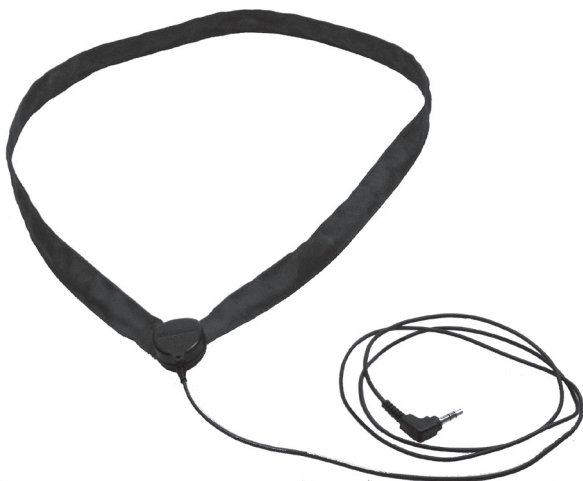Ordering number: 200 22 26

## Neck Loop, 2-channel

With a neck loop, the person using a hearing instrument can receive the sound from different sound sources, such as a CD player. Moreover, this neck loop has two channels, which means that the sound is transmitted from both the left and the right channel of the sound source, and then gives a mono signal in the loop. The neck loop has a 3.5 mm contact and thus can be connected to your CD/MP3 player, music equipment etc.

### Connection

- Turn your hearing aid to the telecoil position.
- Connect the neck loop to the sound source.

### Troubleshooting

Type of fault	Cause	Action
No/weak sound	<ol style="list-style-type: none"> <li>1. The hearing instrument is not in the T position.</li> <li>2. The neck loop is not inserted in a headphone socket.</li> <li>3. The sound source is not switched on.</li> <li>4. The volume is set to low on the sound source.</li> <li>5. The volume is set to low on the hearing instrument.</li> </ol>	<ol style="list-style-type: none"> <li>1. Set the hearing instrument to the T position.</li> <li>2. Connect the neck loop to the headphone socket on the sound source.</li> <li>3. Switch on the sound source.</li> <li>4. Increase the volume on the sound source.</li> <li>5. Increase the volume on the hearing instrument.</li> </ol>

### Technical data

Resistance:	2 x 12.5 $\Omega$ .
Cut-off frequency:	2.6 kHz, -3 dB rel. 1 kHz, "stereo".
Circumference:	74 cm.
Weight:	25 g.
Color:	Black.
Contact:	3-pole angular contact, diameter 3.5 mm on 80 cm cable.

## Cleaning and maintenance

Use a damp cloth to clean the product on the outside. Never use cleaning products or solvents. From time to time check that the cables and connections are intact.

## Repairs

If your product malfunctions, it must be repaired by a qualified technician. Do not attempt to open the case of the device since this would invalidate the warranty. If your product requires service, please contact your hearing care professional for assistance.

## Waste disposal

This product contains electrical or electronic equipment and should be disposed of carefully in the interest of your safety and the environment. Please contact your local hearing care professional concerning disposal of the product.

## General precautions

- Low-energy radiation (e.g. from burglar alarms, room surveillance systems, radio equipment, mobile telephones) will not damage the device. They could, however, momentarily affect the sound quality or create noise in the device.

## Warning –strangulation risk

- Potential risk for strangulation if Neck Loop gets caught in object.
- Always keep children and people with limited mental skills under observation.

**CE** Any issues relating to the EU Medical Device Directive 93/42/EEC should be directed to AB Transistor Sweden.

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